

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\ACAPRIHAN\phantom_20200\ugrant_adni_phantom\unm_cvr_12ch_qc

TA: 7:17 PAT: 2 Voxel size: 3.4x3.4x3.8 mm Rel. SNR: 1.00 USER: ep2d_complex

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

Routine

Slice group 1	
Slices	34
Dist. factor	0 %
Position	R2.3 A20.9 F29.0
Orientation	Transversal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
FoV read	220 mm
FoV phase	100.0 %
Slice thickness	3.8 mm
TR	1500 ms
TE	21 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

MTC	Off
Flip angle	90 deg
Fat suppr.	Fat sat.

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	287
Delay in TR	0 ms
Multiple series	Off

Resolution

Base resolution	64
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	24
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Separate

Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Raw filter	On
Elliptical filter	Off
Hamming	Off

Geometry

Multi-slice mode	Interleaved
------------------	-------------

Series

Series	Ascending
Special sat.	None

System

Body	Off
HEP	On
HEA	On

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	R2.3 A20.9 F29.0
Orientation	Transversal
Rotation	0.00 deg
R >> L	220 mm
A >> P	220 mm
F >> H	130 mm

Physio

1st Signal/Mode	None
-----------------	------

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Starting ignore meas	0
Ignore after transition	0
Model transition states	Off
Temp. highpass filter	Off
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active
Meas[15]	Active
Meas[16]	Active
Meas[17]	Active
Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off
Spatial filter	Off

SIEMENS MAGNETOM TrioTim syngo MR B17

Sequence

Introduction	Off
Bandwidth	2442 Hz/Px
Free echo spacing	Off
Echo spacing	0.49 ms

EPI factor	64
RF pulse type	Normal
Gradient mode	Fast