

SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\GROSENBERG\Ugrant_20294\Ugrant_20294_V3\unm_t2w_final_20181115

TA: 4:26 PAT: 2 Voxel size: 1.0x1.0x1.0 mm Rel. SNR: 1.00 SIEMENS: tse_vfl

Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	On
Start measurements	single

Routine

Slab group 1	
Slabs	1
Position	R0.5 A23.6 F37.6
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	100.0 %
Slice thickness	1.00 mm
TR	4000 ms
TE	427 ms
Averages	1.0
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

Contrast

MTC	Off
Magn. preparation	None
Fat suppr.	None
Water suppr.	None
Restore magn.	Off

Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Allowed
Slice partial Fourier	6/8
Interpolation	Off

PAT mode	GRAPPA
Accel. factor PE	2
Ref. lines PE	32
Accel. factor 3D	1
Matrix Coil Mode	Auto (Triple)
Reference scan mode	Integrated

Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On

Normalize	Off
B1 filter	Off
Raw filter	Off
Elliptical filter	Off

Geometry

Special sat.	None
--------------	------

System

Body	Off
HEP	On
HEA	On

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
Auto Coil Select	Default

Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	R0.5 A23.6 F37.6
Orientation	Sagittal
Rotation	0.00 deg
F >> H	256 mm
A >> P	256 mm
R >> L	176 mm

Physio

1st Signal/Mode	None
-----------------	------

Dark blood	Off
------------	-----

Resp. control	Off
---------------	-----

Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Sequence

Introduction	On
Dimension	3D
Bandwidth	651 Hz/Px
Flow comp.	No
Allowed delay	0 s
Echo spacing	3.46 ms
Adiabatic-mode	Off

Define	Echo trains
--------	-------------

SIEMENS MAGNETOM TrioTim syngo MR B17

Turbo factor	145
Slice turbo factor	2
Echo trains per slice	1
Echo train duration	924
RF pulse type	Fast
Gradient mode	Fast
Excitation	Non-sel.
Flip angle mode	T2 var