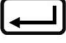


How to use Medtronics CO2 monitor



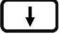
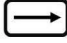
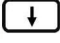

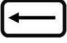

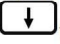

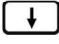


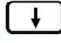
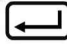
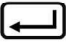

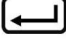
1. Power up the monitor.

- If the “Periodic calibration is required. Please notify technical service” message appears, press  to confirm and continue.

2. Connect the tubings:






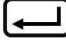
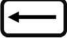
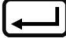
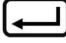

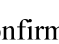
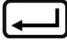
- The orange connector with the golden ring goes into the upper right port (black color) on the right side of the device; Push it into the black port and turn clockwise to lock.
- Once connected well, one can hear the sound of the pump, and the Message Bar shows “CO2 Ready” on the screen.

3. Start saving data:

- Connect the USB drive to the left middle port of the device.
- Press  button to go to the Menu → “Setup” menu will appear → Press  once and then press  multiple times to select the “Report” option → Press  and then press  to select “Store Report” → Press  to confirm → “Active Report” will appear on the left side → Press  and then press  to confirm → The “More” option appeared on the left side of the display → press  to select “More” → Press  once and then  multiple time to select “Realtime continuous waveform” → Press  to confirm → “Choose Device” will appear under “Active Report” on the left side → Press  once and then press  to select “Active Report” → Press  to confirm → “USB 1” will appear and be selected → Press  to confirm → “Save Report” will appear on the left side → Press  to select “Save Report” → Press  to confirm → In the message Bar, “Report transfer to USB in progress” will appear → Press the Home button to return to home screen.

4. Connect nasal cannula and start experiment.

5. Stop saving data

- Press  button to go to the Menu → “Setup” menu will appear → Press  once and then press  multiple times to select the “Report” option → Press  and then press  to select “Store Report” → Press  to confirm → “Active Report” will appear on the left side → Press  and then press  to confirm → “Realtime continuous waveform USB 1” will appear → Press  to confirm → “Stop” will appear under “Active Report” on the left side → Press  once and then press  to select “Stop” → Press  to confirm → Waiting sign will appear and then disappear after a few seconds → In the message Bar, “Report transfer to USB aborted” will appear → Press the Home button to return to home screen.

6. Unplug tubings and power off the monitor